

Before using the device, read the instructions for use, as it contains important information for the patient regarding the use of the device.

Keep the instructions for use so that you can read them again if necessary.

1 / WM Lideal /2017 Atopic skin cream

DESCRIPTION OF MEDICAL DEVICE

The product is intended for use on skin prone to extreme dryness.

The product creates a protective barrier that moisturizes and improves skin elasticity, protecting it from damage, bringing relief in dry conditions that are typical of atopic dermatitis, eczema and skin lesions in the course of psoriasis.

The created protective layer soothes the symptoms of skin dryness (discomfort, tension, itching, burning) and provides the skin with proper hydration and lubrication, protecting it from peeling and cracking

The formula of the product has been designed in such a way as not to burden sensitive and irritated skin.

The use of the product is also recommended for people with skin prone to extreme dryness.

Does not contain PEG, parabens, detergents, including: SLS, SLES

Easy and pleasant application

Does not stick

It can be used on large body surfaces

Product to be used from 1 month of age.

FORMULA

Purified water, petroleum jelly, a mixture of cetyl and stearyl alcohol, mineral oil, a mixture of cetyl and stearyl alcohol ethoxylated with 20 moles of ethylene oxide, phenylethyl alcohol, caprylic glycol

GENERAL INFORMATION and INDICATIONS

The medical device is recommended for:

- alleviating the symptoms of dry skin (discomfort, tension, itching, burning)

associated with:

- AD atopic dermatitis

- skin changes in psoriasis

- eczema

- protection of the affected skin against the influence of external factors

- protection of skin prone to extreme dryness that can cause cracks and damage to the skin.

MECHANISM OF OPERATION OF THE PRODUCT

After application of the product on the skin, a moisturizing protective layer is created, which moisturizes, improves elasticity and soothes the affected skin.

The produced coating also acts as a protective barrier against the influence of irritating factors..

HOW TO USE

The product should be applied to clean skin requiring hydration, prone to cracking or diseased, undamaged skin. Gently spread by patting with your fingertips. Do not rub. When applying the product on the face, avoid the eye area. When applying the product on the face, avoid the eye area. Use 1-3 times a day as needed.

The product is intended for external use only on intact skin

For children up to 12 years of age, use under the supervision of an adult.

WARNINGS AND CONTRAINDICATIONS

Do not use if you are allergic to the ingredients contained in the medical device
Do not use if symptoms of itching or burning occur after using the product, which may indicate an allergy to the components of the medical device, then STOP USING THE PRODUCT.
Avoid accidental contact of the product with the eye mucosa.
The product is intended for pediatric use. When used in children, parent/guardian supervision is required
The use of the product in children under 1 year of age should be consulted with a doctor or pharmacist.
Do not use on the eyelid area.
Keep away from children.
Do not use after the expiry date
Do not use if packaging is damaged or tampered with before first opening
In case of using the device with drugs applied to the skin, consult a doctor or pharmacist
The product is intended for external use only on intact skin

Available pack size

Tube 100g in carton box
Jar 500ml

STORAGE

Store the product in its original packaging.
Store at room temperature, i.e. 15-25°C.
Protect from light and sources of heat.
Keep out of the reach and sight of children.
Do not use the product if the immediate packaging is damaged.
Do not use after the expiry date stated on the packaging..



MANUFACTURER

LABORATOIRE L'IDEAL Parisien Sp. z o.o.

Ul. Ogrodowa 8, 32-064 Niegoszowice

tel./fax +48 12/ 283 88 60

DISTRIBUTOR:



MEDICAL DEVICE



Informacja na opakowaniu

Date of update IFU Edition 05 z dnia 20.04.2023