

Before using the device, read the instructions for use, as it contains important information for the patient regarding the use of the device.

Keep the instructions for use so that you can read them again if necessary.

6 / WM Lideal /2021 Ointment for skin

DESCRIPTION OF THE MEDICAL DEVICE

Device 6 / WM Lideal /2021 Ointment for skin is intended for use on the skin:

- in case of irritations and chafes occurring with diaper dermatitis in infants and small children,
- preventively against the occurrence of diaper dermatitis,
- to relieve the symptoms of discomfort accompanying skin lesions in the course of psoriasis.

The product creates a protective barrier that lubricates the skin, protects against moisture (urine, feces) and irritation, and at the same time soothes the feeling of discomfort (burning, itching) accompanying diaper dermatitis.

The combination of petroleum jelly, linseed oil and Softisan creates a barrier on the skin that protects against the adverse effects of external factors (urine, feces). In addition, it prevents skin dehydration by limiting the evaporation of water from the epidermis in the evaporation process called TEWL (Transepidermal water loss).

Softisan is suitable for use on very sensitive skin, it is non-allergenic. Acting as a film-forming agent, it additionally has a high water-binding capacity and good adhesion to the skin.

Although diaper dermatitis has a factor of increased moisture under the diaper, it is important to ensure proper hydration of the delicate and sensitive skin of the newborn, which reacts very quickly to changes such as: inadequate moisture, changes in pH, contact with irritants.

The main task of the product is protective action to minimize the susceptibility of the immature skin of a newborn, infant or older child to injury as a result of skin exposure to irritants such as urine or feces and the presence of the diaper itself.

In addition, the combination of petroleum jelly, linseed oil and Softisan creates a protective barrier on the skin, thanks to which it soothes symptoms such as tension, itching, burning, excessive flaking associated with excessive skin dryness characteristic of skin lesions in psoriasis. The product helps to restore proper hydration of the skin by stopping the evaporation of water from the affected epidermis. Relieving the symptoms of dryness, it also protects the affected skin from external factors. The use of the product can be a supplement to dermatological therapy in the course of psoriasis.

Product to be used from 1 day of life.

COMPOSITION

White petrolatum, a mixture of diglycerol esters and polyacyladipate, linseed oil.

GENERAL INFORMATION AND INDICATIONS

Medical device 6 / WM Lideal /2021 Ointment for skin is recommended:

- supportive in diaper dermatitis, when the clinical picture is characterized by symptoms dermatitis with irritants such as erythema and swelling, but not in the case of skin damage: papules exudative and erosions,
- protective, creating a delicate protective film/barrier on the skin, which protects the skin against moisture (urine, feces) and irritations,
- soothing, reducing the feeling of burning and itching in diaper dermatitis,
- preventively in order to minimize the formation of skin irritations and chafing,
- soothing in the case of skin lesions occurring in the course of psoriasis in adults,
- soothing in the case of skin lesions occurring in the course of psoriasis in children and infants (only after consulting a doctor).

MECHANISM OF OPERATION OF THE PRODUCT

After application of the product on the skin, a protective barrier is created on its surface. The protective barrier protects the skin from the negative effects of irritants. At the same time, it prevents the loss of water through the epidermis layer, which helps to regain proper hydration of the epidermis and the related natural, physiological barrier of the skin.

HOW TO USE

Apply the product to clean skin that requires protection against irritation, chafing or burns on diseased, undamaged skin. Gently spread by patting with your fingertips. Do not rub. Use 2-3 times a day as needed. In the case of children and infants, the use of the product on psoriatic lesions is allowed only after consulting a doctor.

WARNINGS AND CONTRAINDICATIONS

Do not use in case of hypersensitivity to the ingredients contained in the medical device. Discontinue use if you experience any itching or burning symptoms after using the product, which may indicate an allergy to the components of the product. Avoid accidental contact of the product with the eye mucosa. Keep away from children. Do not use after the expiry date stated on the packaging and the seal of the tube. Do not use if the package is damaged or tampered with before first opening. Use on skin lesions in the course of psoriatic disease in children and infants only after consulting a doctor. Do not use on damaged skin and unhealed wounds. Do not use in the case of diaper dermatitis with exudative papules and erosions. If there is no improvement after using the product for diaper dermatitis, please contact your doctor. The product is intended for external use only on intact skin.

STORAGE

Store the product in its original packaging. Store at room temperature, i.e. 5-30 °C. Protect from light and sources of heat. Keep out of the reach and sight of children. Do not use the product if the immediate packaging is damaged. Do not use after the expiry date stated on the packaging.

Product packaging includes:

50 ml tube or 100 ml tube with instructions for use



MANUFACTURER


LABORATOIRE L'IDEAL Parisien Sp. z o.o.

Ul. Ogrodowa 8, 32-064 Niegoszowice

tel./fax +48 12/ 283 88 60



Wyrób medyczny

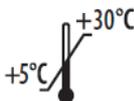

Information on the packaging