

Before using the product, please read the instructions, as they provide valuable details for patients about the product's applications.
Keep the instructions for future reference.

4 / WM Lideal/2021 Mouth ulcer gel

DESCRIPTION OF MEDICAL DEVICE

A specialized medical product for is intended for use in the oral cavity to relieve the lower pain associated with the occurrence of changes, such as aphthae, thrush, minor injuries. Product by creating a layer protective, relieves pain, is a barrier against infectious agents and irritations for the altered places in the oral cavity. The applicator enables easy and hygienic application of the product, which ensures high comfort of use.

INGREDIENTS

Purified water, propylene glycol, polyvinylpyrrolidone, polyoxyethylene with 40 moles of ethylene oxide hydrogenated castor oil, xanthan gum, potassium sorbate, sodium hyaluronic acid, glycyrrhizinic acid, sodium saccharinate, benzalkonium chloride.

GENERAL INFORMATION AND INDICATIONS FOR USE

The product is recommended for:

- aphthous ulcers,
- aphthous stomatitis,
- thrush,
- minor injuries, including those caused by braces and poorly fitting dentures

HOW THE PRODUCT WORKS

After application within the oral cavity, the gel produces a moisturizing protective layer that stays in place, which soothes and relieves discomfort caused by lesions in the mouth. The layer produced covers lesions in the oral cavity, shielding them from infectious agents and mechanical injuries.

DIRECTIONS FOR USE

Wash your hands thoroughly before applying the product.

Apply topically.

1. Remove the protection from the applicator.
2. Place the neck of the applicator inside the mouth.
3. Apply 1 or 2 drops of spot gel to a lesion in the mouth.
4. Avoid direct contact of the applicator with the lesion
5. Remove the applicator and secure it with the cap.

Adults and children:

Apply 3-4 times a day or more if necessary.

It is advised to refrain from eating for about 1 hour after applying the product.

The product can be used regularly many times a day, in varied amounts, depending on your needs.

For health and epidemiological purposes, one product cannot be used by more than one user.

The applicator can be washed with soap and water, rinsed several times, dried, and reused.

WARNINGS & CONTRAINDICATIONS

- Do not use in case of hypersensitivity to the ingredients contained in the medical device.
- Discontinue use if symptoms of itching or burning appear after using the product, which may testify to an allergy to the ingredients of the product.
- If the condition of the oral mucosa deteriorates or symptoms worsen, consult your doctor expert.
- Accidental contact of the product with the eye mucosa should be avoided.
- Keep away from children.
- Do not use after the expiry date stated on the packaging and on the tube seal.
 - Do not use if the packaging is damaged or tampered with before first opening.
- Do not use the product if the applicator is damaged.
- For hygienic reasons, it is recommended to use one product by one patient.
- The product is intended for use in adults and children over 1 year of age.
- In children, the product should be used under the supervision of an adult.

STORAGE

Store the product in the original packaging.
Store the product at room temperature of 15-25°C.
Keep away from light and heat.
Keep out of the reach and sight of children.
Do not use the product if the original packaging is damaged.
Do not use after the expiration date stated on the packaging.

The packaging includes:



MANUFACTURER

LABORATOIRE L'IDEAL Parisien Sp. z o.o.

Ul. Ogrodowa 8, 32-064 Niegoszowice

tel./fax +48 12/ 283 88 60

DISTRIBUTOR:



Medical device



Information on the packaging

Date of instruction update Revision 02 of 15/02/2022