

Before using the product, please read the instructions, as they provide valuable details for patients about the product's applications.
Keep the instructions for future reference.

Scar gel

DESCRIPTION OF MEDICAL DEVICE

Advanced medical aid used for the treatment of previously healed scars of different origins. The gel has a smoothing and soothing effect on scar tissue, improves its appearance by softening and evening out the colour, and reducing the discoloration. The product reduces itching and the tangible tension of the skin.

Absorbs quickly
Does not contain PEG, parabens, detergents (including SLS, SLES)
Non-sticky
Safe for the skin
Provides all-day protection

INGREDIENTS

BioGenic Gensil-210L: polydimethylsiloxane, water, copolymer of dimethicone and vinylated dimethicone, butylene glycol, sodium hyaluronic acid, phenoxyethanol, hydroxypropyl bispalmitamide mea; polydimethylsiloxane, trimethylsiloxysilicate, dimethicone copolymer, silicon dioxide

GENERAL INFORMATION AND INDICATIONS FOR USE

The product is recommended for:

- 1) enhancing the treatment of scars of various origins, including after:
 - burns,
 - injuries (abrasions, cuts),
 - surgeries (C-section),
 - surgical treatment
 - acne
- 2) as the scar's protective layer against rubbing and sunlight

HOW THE PRODUCT WORKS

After application on the skin, the product produces a moisturizing protective layer, that stays in place, which in turn significantly reduces friction and provides long-term protection against mechanical damage to the scar as well as abrasions and irritations. The product, combined with a scar massage, loosens the scar tissue and reduces it.

DIRECTIONS FOR USE

Apply the product on the scars after the wound has healed completely and the scabs fall off. Apply directly to the scar, starting from the centre of the scar to its edge. Perform a circular scar massage until the substance has been fully absorbed. The product can be used regularly many times a day, in varied amounts, depending on your needs, but not more than 4 times a day.
For external use only.

WARNINGS & CONTRAINDICATIONS

Do not use if you are allergic to the product's ingredients
If you experience itching or stinging after using the product, this may mean you are allergic to the product's ingredients; in that case, STOP USING THE PRODUCT.
Avoid accidental contact with the mucous membrane of eyes and mucous membranes
Do not use on damaged skin or unhealed wounds.
Do not use other products directly after applying the gel to the scar.
Keep away from children.
Do not use after expiration date.
Do not use if the packaging is damaged or tampered with before first opening
The use of silicone gel can cause an MRI artifact to appear and be confused with a metallic foreign body.
This warning should be given to MRI professionals.

The product is intended for external use only on undamaged skin

STORAGE

Store the product in the original packaging.
Store the product at room temperature of 5-30°C.
Keep away from light and heat.
Keep out of the reach and sight of children.
Do not use the product if the original packaging is damaged.
Do not use after the expiration date stated on the packaging.

The packaging includes:

Tube 15ml , 20 ml, 30 ml, 50 ml




MANUFACTURER

LABORATOIRE L'IDEAL Parisien Sp. z o.o.
Ul. Ogrodowa 8, 32-064 Niegoszowice
tel./fax +48 12/ 283 88 60

DISTRIBUTOR:



Medical device

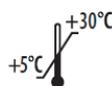

Information on the packaging

Date of instruction update Revision 04 of 30.04.2023