

Before using the product, please read the instructions, as they provide valuable details for patients about the product's applications.

Keep the instructions for future reference.

3/WM Lideal/2021 Teething gel

DESCRIPTION OF MEDICAL DEVICE

A medical device, teething gel is intended for use on the gums as an auxiliary to alleviate discomfort during the period of teething in children.

The product is a gum barrier against irritating factors.

The applicator has silicone insets that allow for hygienic massage of the gums

It does not contain sugar, alcohol, anesthetic substances that can cause side effects if swallowed.

A medical device with a raspberry flavor.

For use from 3 months of age

INGREDIENTS

Aqua, Glycerin, Xylitol, Hydroxyethylcellulose, Sodium Hyaluronate, Aroma, Aloe Barbadensis Leaf Juice, Ammonium Glycyrhizate, Potassium Sorbate, Sodium Benzoate, Crocus Sativus Flower Extract, Sodium Hydroxide

GENERAL INFORMATION AND INDICATIONS FOR USE

The product is recommended for:

- 1) alleviating the pain caused by teething
- 2) as a protective layer for the gums against infectious agents and irritations during teething
- 3) reduces irritation and redness of the gums where teeth are erupting
- 4) reduction of gum swelling during teething

HOW THE PRODUCT WORKS

After the product is applied, a moisturizing film is created on the gum surface protective, which soothes the pain associated with teething, reduces swelling and protects the gums against mechanical irritation. Due to its properties, the sodium hyaluronate contained in the product soothes discomfort and accompanying pain during teething in children.

DIRECTIONS FOR USE

Before using the product, wash your hands thoroughly. Apply topically. The product can be applied on red and irritated gums with a clean finger or applicator. Apply just enough gel to cover the affected area of the gums. After applying the product, gently massage the gums with the applicator. It is recommended to use the product after a meal or before going to bed to ensure its effective operation. The product can be used several times a day as needed, but no more than 4 times a day. The product is intended only for use on the gums during teething. After application, clean the applicator of any gel residue that could make re-application of the product difficult. After each use, put the protective cap on the applicator.

WARNINGS & CONTRAINDICATIONS

- Do not use in case of hypersensitivity to the ingredients contained in the medical device.
- Discontinue use if symptoms of itching or burning occur after using the product.
- Accidental contact of the product with the eye mucosa should be avoided.
- Keep away from children. The product is not a toy.

- Do not use after the expiry date stated on the package.
- Do not use if the packaging is damaged or tampered with before first opening.
- Before application, check that the applicator is complete and that no part of it has become disconnected.
- Do not use the product if the applicator is damaged.
- The product should not be taken as a product to be swallowed.
- The product should only be applied to a child's gums by an adult.
- For hygienic reasons, it is recommended that one product be used by one patient.
- The product is intended for children from 3 months of age, only for use on the gums during teething

STORAGE

Store the product in the original packaging.
Store the product at room temperature of 15-25°C.
Keep away from light and heat.
Keep out of the reach and sight of children.
Do not use the product if the original packaging is damaged.
Do not use after the expiration date stated on the packaging.

The packaging includes: tube 15ml



MANUFACTURER

LABORATOIRE L'IDEAL Parisien Sp. z o.o.
Ul Ogródowa 8, 32 – 064 Niegoszowice tel./fax
+48 12/ 283 88 60

DISTRIBUTOR:



Medical device



Information on the packaging

Date of instruction update Revision 03 of 24/052024